



Cleaning Instructions

Routine Care/Cleaning

If the mat is not heavily or visibly soiled, you may not need to launder the mat. Always follow hospital protocol for cleaning mattresses.

To wipe down the mat with disinfectant between patient uses, follow these steps:

1. Place the mat on a clean, flat surface or hang it from the wall.
2. Apply disinfectant solution by spray, solution or pre-soaked wipes.
3. Make sure that you disinfect all mat surfaces. Pay particular attention to the high contact areas, such as the pull handles and patient safety straps.
4. Follow disinfectant's instructions for appropriate contact time and rinsing requirements.
5. Allow the mat to dry before returning to use.

Note: Reference your PPS Glide Operations manual for a listing of recommended disinfectants.