


Instructions

How to use the CADDY

Setup

Attach the Caddy to the head of the patient bed under the mattress. Ensure it is secure using the Velcro strips.

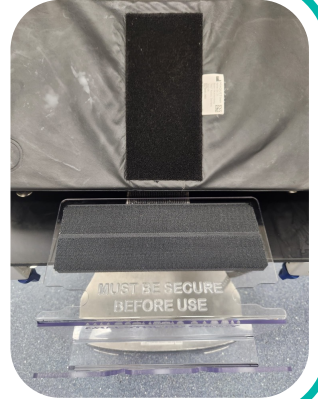

Ways to use the Caddy



Circuit Tubing

Can be secured using the side slots



Blood Pressure Cuff

Can be stored in the groove



Bed Controller

Can be hung from the front panels

Instructions

How to use the CADDY




Ventilation Bag

Can be secured using the front slots



Clip Modules

Can be secured onto the side of the front panel. Spare cable can be rolled up and secured in the slots.

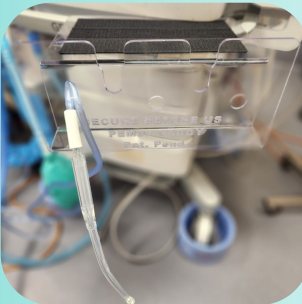

Suction Tubing

Can be folded and secured in the front 3 holes



Monitoring Modules

Can be secured in the central groove. Spare cable rolled and stored in the front panel slots.